


This is a chairside guide to have in your surgery during your OVC procedures. Please ensure you have read the full instructions for use prior to using this guide. Visit [www.rhondium.com/instructions-for-use](http://www.rhondium.com/instructions-for-use) or scan this QR for the most up to date instructions.



## 1) Prep

1.5mm reduction with bevelled margins

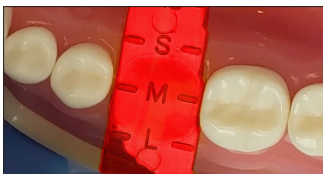

## 2) Measure

Use nearest fit OVC



## 3) Open case

OVC is in the lid



## 4) Replica checking

Check occlusal clearance



## 5) Place matrix

Seal the bottom of embrasure with a wedge if needed



## 6) Burnish

Burnish for desired shape, make sure matrix band touches contacts




## 7) Apply separator

Get a full coverage of prep surface and air dry



## 8) Press OVC down

Align, get patient to bite down and clean up excess material

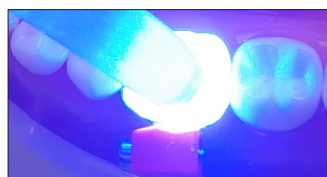


## 9) Cure

20s each cusp (1000mW/cm<sup>2</sup>) and remove OVC



## 10) Clean up/polish

Light cure fully. Remember to completely remove separating agent from OVC

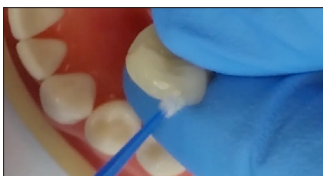

## 11) Add flowable

Apply small amount of bonding agent and flowable to contacts for a tight fit



## 12) Bond


Use a dual cure luting cement

This is a chairside guide to have in your surgery during your OVC procedures. Please ensure you have read the full instructions for use prior to using this guide. Visit [www.rhodium.com/instructions-for-use](http://www.rhodium.com/instructions-for-use) or scan this QR for the most up to date instructions.



## 1) Prep

1.5mm reduction with bevelled margins

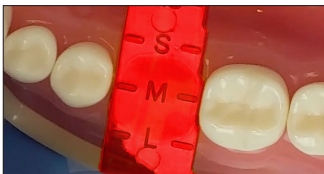

## 2) Measure

Use nearest fit OVC



## 3) Open case

OVC is in the lid



## 4) Replica checking

Check occlusal clearance



## 5) Place matrix

Seal the bottom of embrasure with a wedge if needed, etch and bond prep



## 6) Elevate box

For deep boxes (3mm or more) build up with composite




## 7) Place wedges

Pull hard, allow ligaments to stretch and pull hard again



## 8) Burnish

Burnish matrix band. Use flowable to hold matrix against adjacent teeth

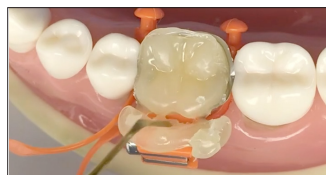


## 9) Press OVC down

Align, get patient to bite down and clean up excess material




## 10) Spot cure

Spot cure 3-5s and remove more excess

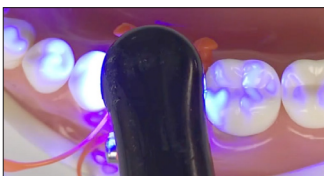

## 11) Final cure

20s each cusp (1000mW/cm<sup>2</sup>) and remove matrix and wedges

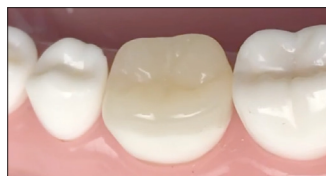

## 12) Adjust and polish

Trim and polish OVC and check bite and adjust if necessary